

Protevo™ APC

Convertible Alternating Pressure Air Surface



Convertible Prevention and Treatment



Protevo APC is an advanced pressure redistribution mattress that provides flexible therapy to help you meet the wound care needs of your patient. Used without the pump, Protevo APC provides pressure redistribution through self adjusting air cells. Easily convert the surface with an optional pump to provide alternating pressure therapy.

Protevo APC – convertible alternating pressure air surface

Highlights

- Hybrid visco-elastic foam increases patient comfort and helps reduce the occurrences of pressure ulcers
- Eight air cylinders absorb and displace patient weight
- Mattress can be used without pump as a static air surface
- Alternating pressure therapy provided in 10 minute cycle time
- Specially designed to help protect the scapula, sacrum, trochanter and heels
- 500 lbs maximum patient weight

The Protevo APC mattress can be used with any flat deck frame. When used with a LINET bed, the Protevo APC mattress works in combination with the Ergoframe® design of the bed to help maintain patient position, neutralize shear and minimize pressure in the sacral area. This leads to improved patient comfort and easier and safer nursing care.

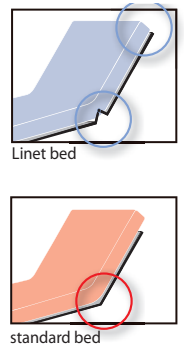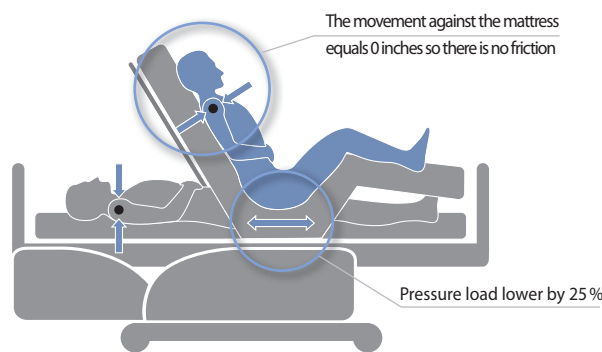

Protevo APC Construction

- 7 inch mattress height
- Air Cells:
 - Eight cells directly beneath visco-elastic comfort topper
 - 4-1/2 inch air cell height
 - Provide static pressure redistribution without a pump
 - Alternate in A-B system when connected to a pump to provide alternating pressure therapy
 - 10 minute alternating pressure cycle
- Side bolster foam for support
- Topper:
 - 2-1/2 inch hybrid visco-elastic foam
 - Allows for quicker bounce back than traditional visco
 - Less heat build up
 - Mimics comfort of traditional visco
 - Spans entire length and width of mattress
- 4-way stretch Dartex cover (replaceable)
- Non-slip mattress bottom
- 5 degree heel slope